

# Mission™ UTI Urinary Tract Infection Test Strips



## Effective Testing and Monitoring

- Perform a simple urine test comfortably and privately at home for possible UTI Urinary Tract Infection
- Help doctor detect and monitor UTI frequently before it becomes serious

## Convenient

- Easy to test and read
- Instant results after 2 minutes

## Accurate


- Obtain reliable results by using the same strips that your doctor uses

## Affordable


- Inexpensive testing without the need of laboratory instruments or home equipment

## Multiple Packaging Options and Long Shelf Life


- Canister Packaging
  - Available in 25, 50, 100 and 150 strips per kit
  - 2 year shelf life for strips in unopened canisters which offers cost savings and convenience for high volume testing
  - 3 month shelf life for strip in opened canisters
- Pouch Packaging
  - Individually packaged strips with 3 or 6 strips with 1 color chart per kit
  - Unique packaging maintains 2 year shelf life for all strips in the kit compared to 3 months for remaining strips in opened canister

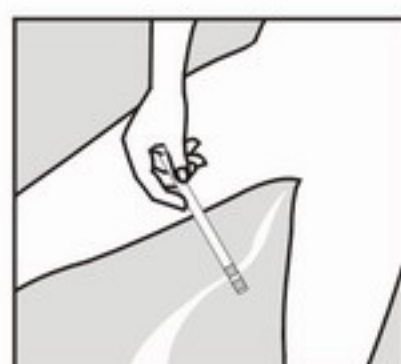

Step 1: Hold the strip downward with the test pads pointing into the urine stream for 1-2 seconds

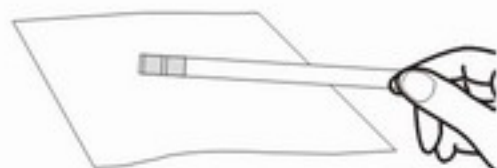

Step 2. Remove excess urine

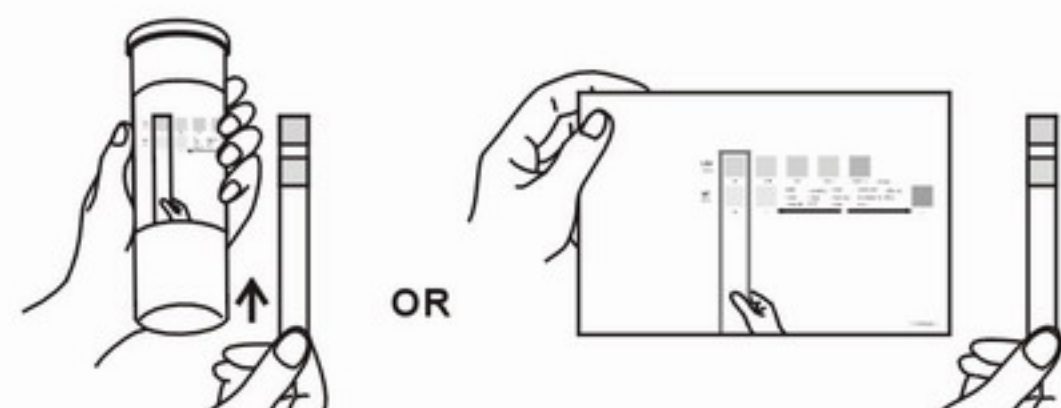

Step 3. Compare the test pad(s) to the color blocks on the color chart

Catalog No.	Test Name	Intended Use	Strips per Canister	Individually Pouched Strips per Kit	Parameter List	
					NITRITE (NIT)	LEUKOCYTES (LEU)
U031-021	UTI Urinary Tract Infection	OTC	3 or 6	3 or 6	*	*

Cleared for US 510k and CLIA Waived

CE Marked for sale in the European Community 

Contact for distribution and custom manufacturing (OEM) opportunities



ACON Laboratories, Inc., 4108 Sorrento Valley Boulevard, San Diego, CA 92121, U.S.A. • Tel: 1-858-875-8000 • Fax: 1-858-535-2035 • E-mail: [info@aconlabs.com](mailto:info@aconlabs.com)  
Please visit our website for details: [www.aconlabs.com](http://www.aconlabs.com)