

# Accurate Results for Every Animal. *As they should be.*

# CentriVet™

Blood Glucose & Ketone Monitoring System  
for Small & Large Animals

## A different kind of meter...

The CentriVet™ GK Blood Glucose & Ketone Monitoring System is a **dual meter** that measures both blood glucose and blood  $\beta$ HBA, giving you accurate results in seconds. It has never been easier to take control of your pet's health!

- ✓ Species specific code chips for testing multiple animals
- ✓ Glucose results in 5 seconds, Ketone results in 10 seconds
- ✓ Strip Port Ejector for convenience and safety
- ✓ 3,000 test battery life
- ✓ Big, bold screen for easy viewing
- ✓ Portable, handheld device designed for multiple settings
  - Veterinarian's office
  - Farm Use
  - Home use
  - Emergency animal hospital
- ✓ 450 test memory with date, time, and test type



## Technology Used By Vets, Now Available for Home Use.

- ✓ Two tests in one meter
- ✓ Priced for profit
- ✓ Clinically proven accuracy\*
- ✓ Multiple species



You love them like family,  
treat them like family.

Glucose and Ketone Testing  
Available for:

Canine



Feline




ACON

www.centrivet.com

# CentriVet™ Blood Glucose Test Strip


## Accuracy Study with Feline/Canine Blood

**Feline Glucose Accuracy Study**  
CentriVet™ Blood Glucose & Ketone Monitoring System

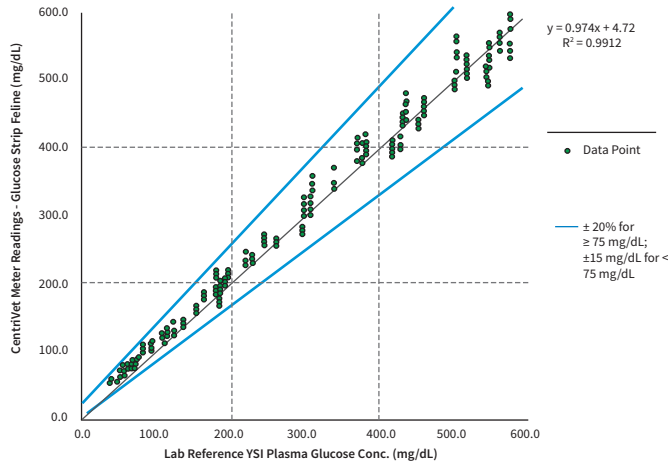

System Accuracy Results for Glucose Concentrations $\geq 75$ mg/dL and $< 75$ mg/dL	
Within $\pm 15\%$ of $\pm 15$ mg/dL	Within $\pm 20\%$ of $\pm 15$ mg/dL
357/360 (99.2%)	359/360 (99.7%)

**Canine Glucose Accuracy Study**  
CentriVet™ Blood Glucose & Ketone Monitoring System

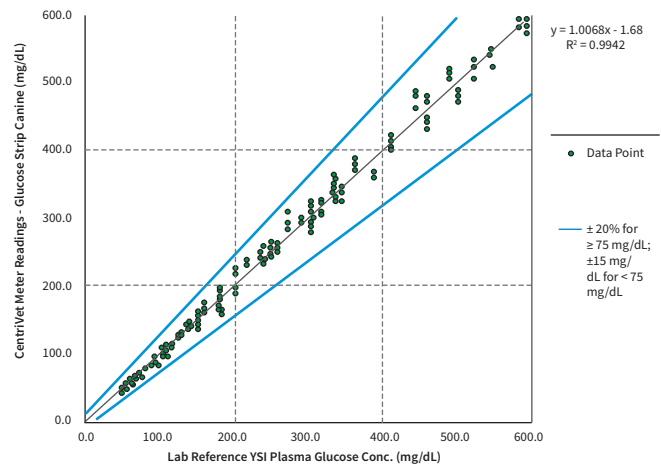

System Accuracy Results for Glucose Concentrations $\geq 75$ mg/dL and $< 75$ mg/dL	
Within $\pm 15\%$ of $\pm 15$ mg/dL	Within $\pm 20\%$ of $\pm 15$ mg/dL
356/360 (98.9%)	360/360 (100%)

## Full Specifications

Feature	Specification
Sample Size	Glucose: 0.8 $\mu$ L
	$\beta$ -Ketone: 1.2 $\mu$ L
Test Time	Glucose: 5 seconds
	$\beta$ -Ketone: 10 seconds
Test Range	Glucose: 0.6 – 33.3 mmol/L (10 – 600 mg/dL)
	$\beta$ -Ketone: 0.0 – 8.0 mmol/L
HCT Range	20-60% for both Glucose and $\beta$ -Ketone
Memory Storage	450 blood glucose and ketone records with result type, date, and time
Test Averaging	7, 14, 30, 60, and 90- day averaging for glucose, before and after meal markers for glucose testing
Battery Life	3,000 tests
Operating Temperature Range	5 - 45°C (41-113°F)
Operating Humidity Range	10 – 90%
Automatic Shutoff	2 minutes of inactivity
Power Source	2 CR 2032 3.0V coin cell batteries

## Ordering Information

Product	Contents	Item Code US	Reference No. International
CentriVet™ GK Blood Glucose & Ketone Meter Kit	Meter, Batteries, Carrying Case, User's Manual, Reference Materials, Warranty Cards	781211	GK114-13C
CentriVet™ Blood Glucose Test Strips	50 Strips/vial	781241	GK134-12C
CentriVet™ Glucose Control Solution	2 x 2 mL bottles	781232	GK124-13C
CentriVet™ Blood Ketone Test Strips	25 Strips/vial	781242	GK134-10C
CentriVet™ Ketone Control Solution	2 x 2 mL bottles	781231	GK124-12C

\*Additional data and instructions can be found in the package insert

ACON Laboratories Inc., San Diego, CA, U.S.A.

PN 2160096002 Rev. C

[www.centrivet.com](http://www.centrivet.com)

