


On-Call Pro[®]


Blood Glucose Monitoring System

How to Perform a Blood Glucose Test


1. Wash hands with soap and warm water and disinfect the puncture site with a topical skin antiseptic. Dry hands and test site thoroughly.

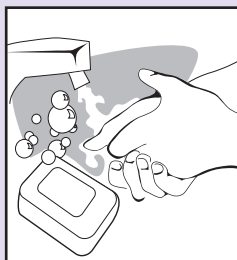


2. Wear appropriate protective gear as recommended by your institution. Wear a new pair of clean disposable gloves before testing each patient.

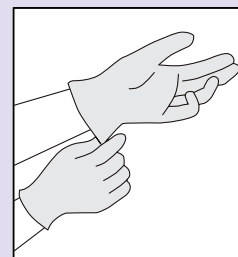

3. Obtain a sufficient blood sample from the patient by using a single-use, auto-disabling safety lancet.

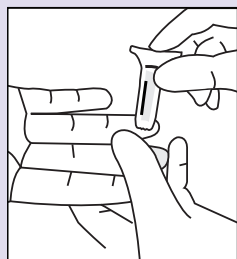


4. Insert a test strip into the strip port. After running a display check, the meter will enter test mode. The display will show the strip icon with the blood sample icon blinking.

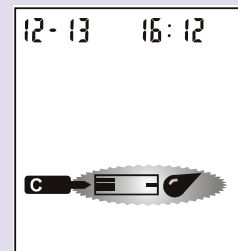


READY TO TEST


5. Touch the blood sample to the sample tip at the end of the test strip.

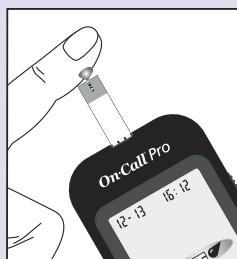


6. The meter will count down from 3 to 1 and then display the result.

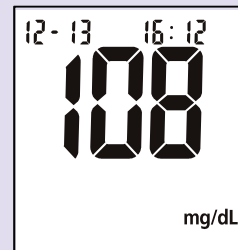

7. Slide forward the strip ejector and discard the used test strip in a biohazardous container.

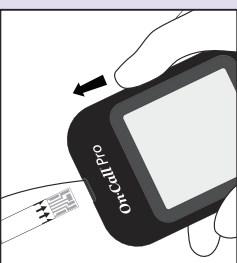

8. Record the results in accordance with your institution's requirements and compare them to the target goals set by the treating physician.

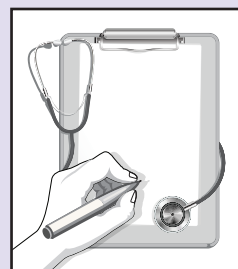

ACON[®]

©2016 ACON Laboratories, Inc. San Diego, CA 92121
PN 2160102001 5/16