



Pregnancy Test

Distinct Packaging Dimensions: 19.2 x 6 x 2.2 cm

Competitively Priced
Quick Results

Highly Reliable and Accurate
– Comparable Performance to Market Leaders

AVAILABLE FOR OEM OPPORTUNITIES

Offering customers an affordable and reliable testing option.



Pregnancy Test

Human chorionic gonadotropin (hCG) is a glycoprotein hormone produced by the developing placenta shortly after fertilization. The test can be used on the first day of the expected period, and any day after that.


General Information

The Distinct Pregnancy Test is a rapid chromatographic immunoassay for the qualitative detection of human chorionic gonadotropin in urine to aid in the early detection of pregnancy.

- Detection level: 25 mIU/mL
- Specimen: Urine
- Test time: 3 minutes


- **SHELF LIFE AND STORAGE TEMPERATURE:** 24 months at 2-30°C
- **CROSS REACTIVITY:** No cross reactivity with 300 mIU/mL LH, 1000 mIU/mL FSH and 1000 µIU/mL TSH

Distinct® Features

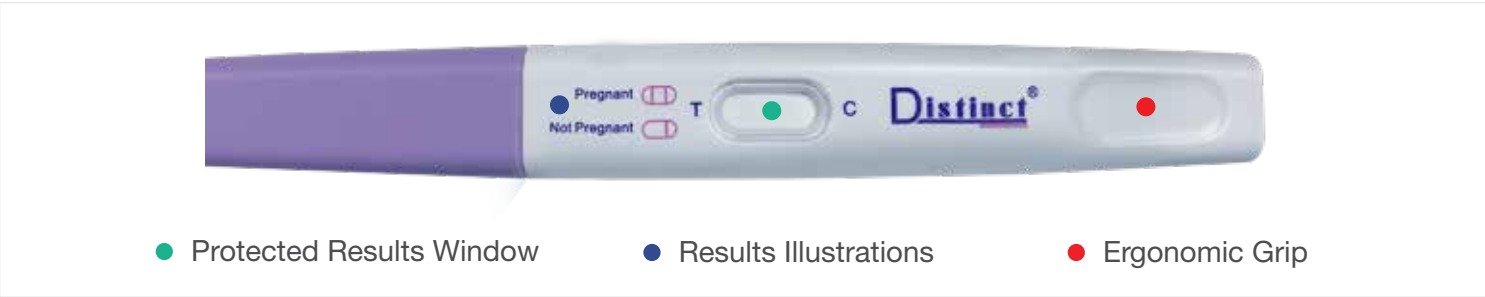

Test Procedure and Interpretation


- ### Midstream
- Hold the device by the grip with the absorbent tip pointing downward in your urine stream for 5-10 seconds, or dip the absorbent tip into your collected urine sample for at least 10 seconds.
 - Read result at 3 minutes

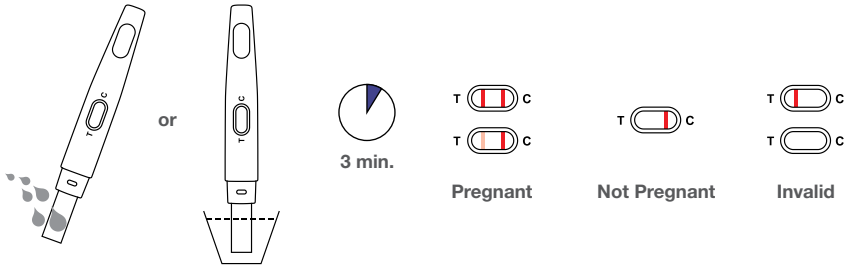

Ordering Information



PRODUCT NAME	CATALOGUE	FORMAT	SPECIMEN	SENSITIVITY	PACKAGE
Midstream Pregnancy Test	L031-20162	Midstream	Urine	25 mIU/mL	1 Test/Kit, 2 Tests/Kit



aconlabs.com

ACON Laboratories, Inc.
 10125 Mesa Rim Road, San Diego, CA 92121, U.S.A.
 Tel: 1.858.875.8000
 Fax: 1.858.200.0729
 Email: info@aconlabs.com