



Distinct Early Detection Pregnancy Test Procedure

If you are testing before your expected period, use your first-morning urine, as it contains the most pregnancy hormone. If testing on the day of or any day after your expected period, you can test at any time of the day.


1. Remove the test from the pouch and remove the Purple Cap. You can perform the test by either of two ways:

In your urine stream: Hold the device by the Thumb Grip with the Absorbent Tip pointed downwards directly in the urine stream. Keep the Absorbent Tip in the stream for 5 -10 seconds.

In your collected urine sample: Collect a sample of your urine in a clean, dry container. Hold the device by the Thumb Grip, with the Absorbent Tip pointed downwards, into the collected urine sample. Keep the tip submerged for 5-10 seconds.



Urine Stream

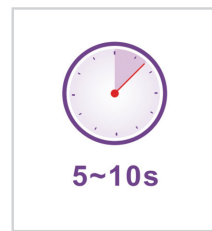

5~10s

OR



Collected Sample

2. After sampling, immediately replace the purple cap over the absorbent tip. Lay the test on a flat surface while results develop.
3. Results will develop over the course of 3 minutes. Do not interpret the results after 10 minutes. A 'Pregnant' result may be seen as early as 1 minute after testing. **You must wait 3 full minutes to confirm a 'Not Pregnant' result.**



Pregnant



Pregnant



Pregnant



Not Pregnant



Invalid



Invalid



aconlabs.com

ACON Laboratories, Inc.
San Diego, CA 92121, U.S.A.
Tel: 1.800.838.9502
Fax: 1.858.200.0729
Email: info@aconlabs.com

