



# Distinct Early Detection Digital Pregnancy Test Frequently Asked Questions (FAQ)

## Q: How does the test work?

**A:** The Distinct Digital Pregnancy Test detects the hormone, existing in your urine, that your body produces during pregnancy (human Chorionic Gonadotropin - hCG). The amount of pregnancy hormone in your body increases as pregnancy progresses.

## Q: How soon after I suspect that I am pregnant can I take the test?

**A:** You can use the Distinct Digital Detection Pregnancy Test up to 6 days early (5 days before the day of your expected period). If you test before your expected period and get a "Not Pregnant" result there is still a chance that you may be pregnant. The chart below shows results from clinical testing done with early pregnancy samples.

Testing Early						
Number of days before the expected period	0	1	2	3	4	5
Percentage of samples giving a "pregnant" result	> 99%	> 99%	99%	94%	89%	72%

## Q: Do I have to test with my first morning urine?

**A:** If you are testing before your expected period, we recommend that you test with your first morning urine as it is usually the most concentrated of the day and would have the most pregnancy hormone (hCG) in it. If you are testing on the day of, or after your missed period, you can test at any time of the day. Please avoid excessive fluid intake before testing as it could dilute the urine, and lead to inaccurate results.

## Q: How accurate is the test?

**A:** A clinical evaluation was conducted comparing the Distinct Early Detection Pregnancy Test to a market leading pregnancy test. The results demonstrated > 99% overall accuracy .

## Q: Can any conditions affect the result?

**A:** Medications containing hCG can affect the result.

Ectopic pregnancy and recent pregnancy, even if not carried to full term can cause misleading results.

If you are in, or approaching, menopause you may obtain a false pregnant result even though you are not pregnant

If you get unexpected results discuss them with your doctor.



aconlabs.com

ACON Laboratories, Inc.  
San Diego, CA 92121, U.S.A.  
**Tel:** 1.800.838.9502  
**Fax:** 1.858.200.0729  
**Email:** info@aconlabs.com

