

Cholesterol Management Software User's Guide

For data transfer to a Personal Computer (PC).

For professional and self-testing in vitro diagnostic use only.

INTENDED USE

These instructions define the format and timing for data transmission from the Cholesterol Meter. Data transferred from the meter includes ID, Date Time, CHOL, HDL, TRIG, LDL, CHOL/HDL and CHD. The intended use is to provide information required to transfer data to an external computer with suitable PC software.

SYSTEM REQUIRED

The following are minimum requirements for CHOL Management Software operation:

Operating System: Windows 7 (32 bit / 64 bit); Windows XP (32 bit); Windows 10 (32 bit / 64 bit)

Processor: Pentium 4-class processor or equivalent

Memory: 512 MB

Hard Disk Space: 100 MB (depends on amount of data to be managed)

Connection Port: Available USB port (USB to Serial port adapter accessory is available)

Display: At least 1024 × 768

MATERIALS REQUIRED

- Cholesterol Meter
- USB Cable

Note: To make use of this software, you need a USB Cable from **Acon**.

DIRECTIONS FOR USE

Installing the Cholesterol Management Software

Note: Please be sure to install the Cholesterol Management Software under administrator's privilege.

1. Download the Cholesterol Management Software to your PC. Double click "CHOL_Management_Software_Setup_v1.2.0.exe" to start the installation process.



2. The Setup Wizard will display as shown below. Click **Next>**.




3. The License Agreement will be displayed as shown below. If you agree with the terms, select ***I accept the agreement*** option and click ***Next>***.




Cholesterol Management Software User's Guide

4. The Destination Location will be displayed as shown below. The default folder is "CHOL Management Software". It is suggested not to change default settings. Click **Next>**.

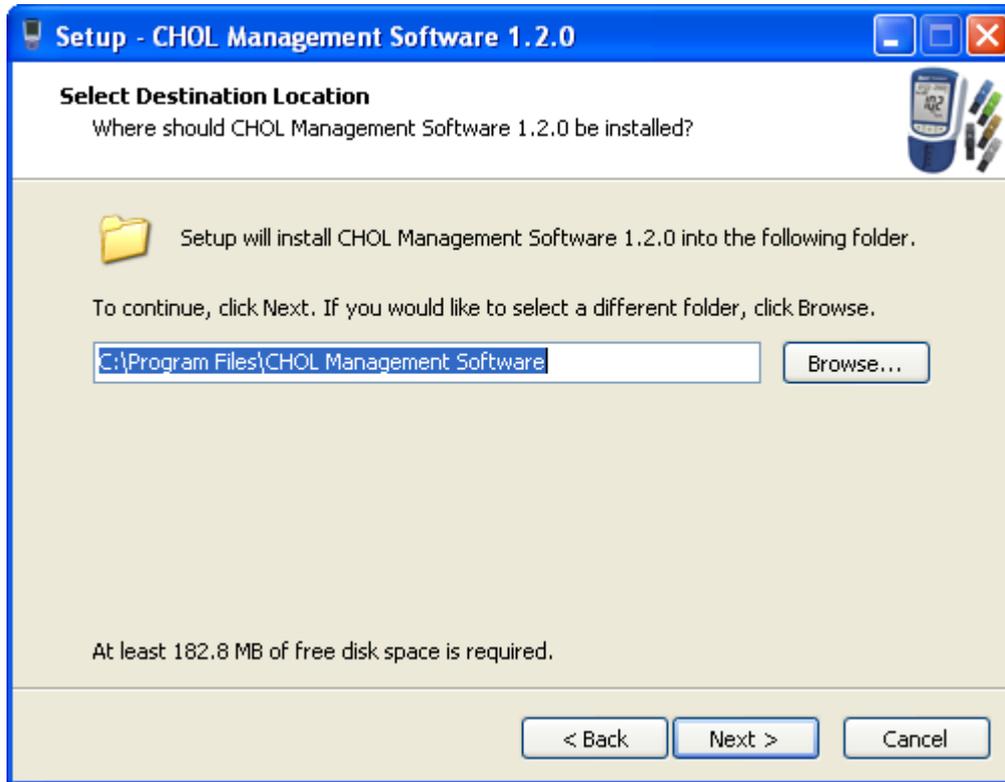


5. Select **Create a desktop icon**, then click **Next>**.

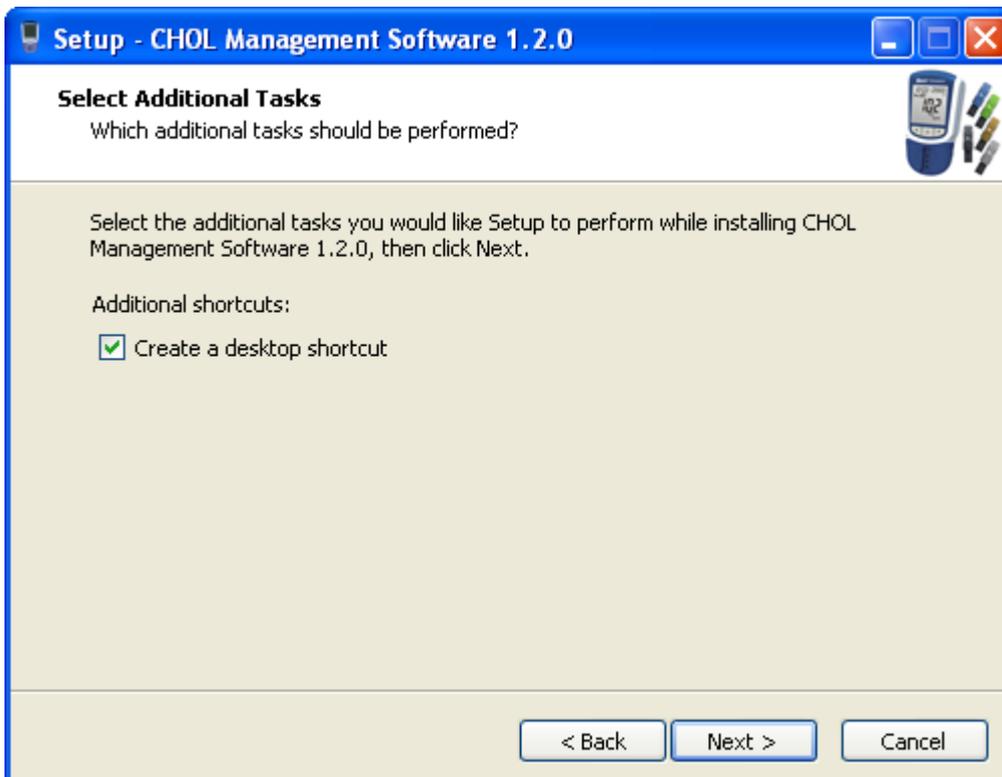


Cholesterol Management Software User's Guide


6. Before the installation files are installed, the screen shown below will be displayed. Click **Install** to continue with the installation process. Click **<Back** to modify or review any of the settings.

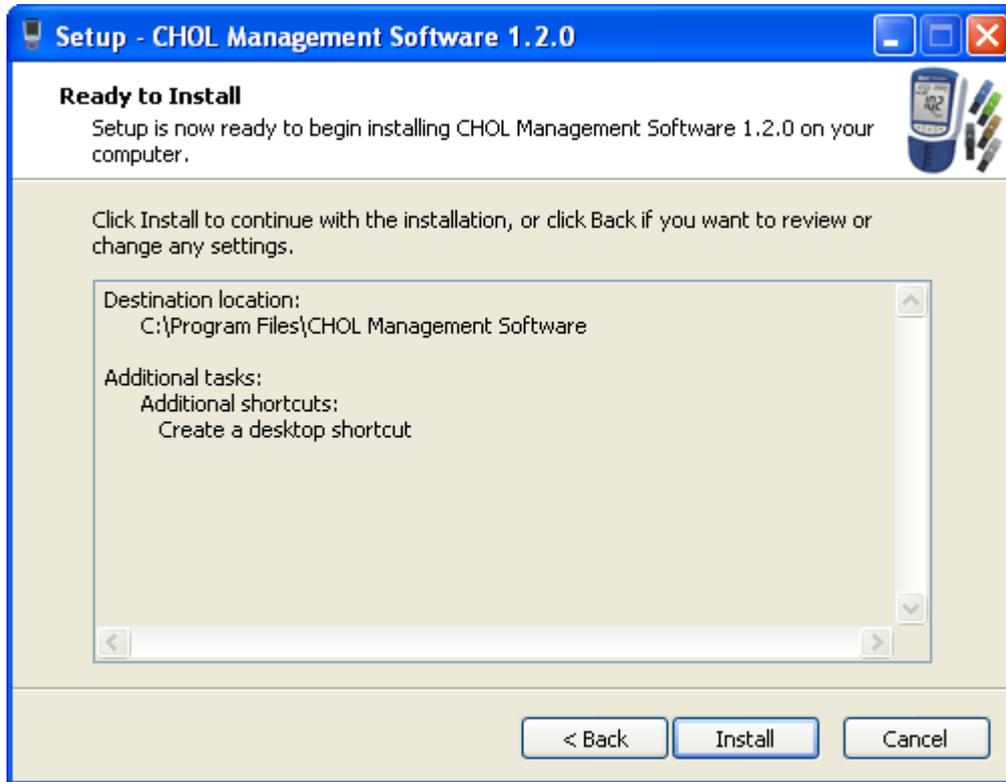


7. The following screen will be displayed, showing the progress of the installation.

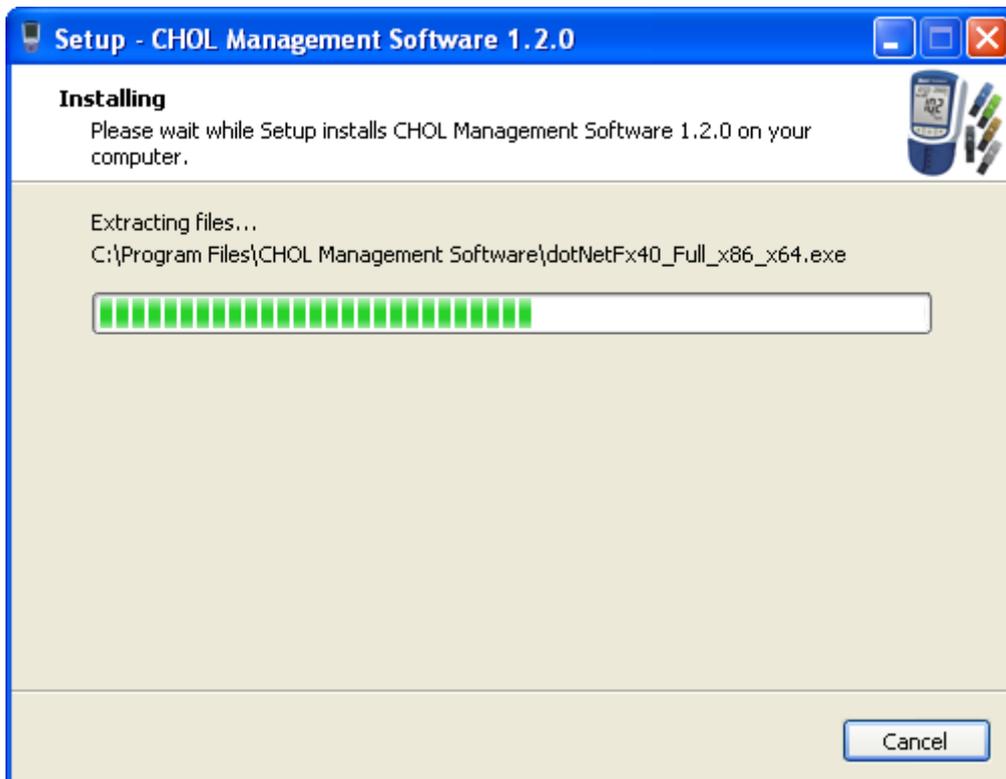

8. The *.NET Framework* will be installed in your computer. Please select ***I have read and accept the license terms*** and click ***Install***.

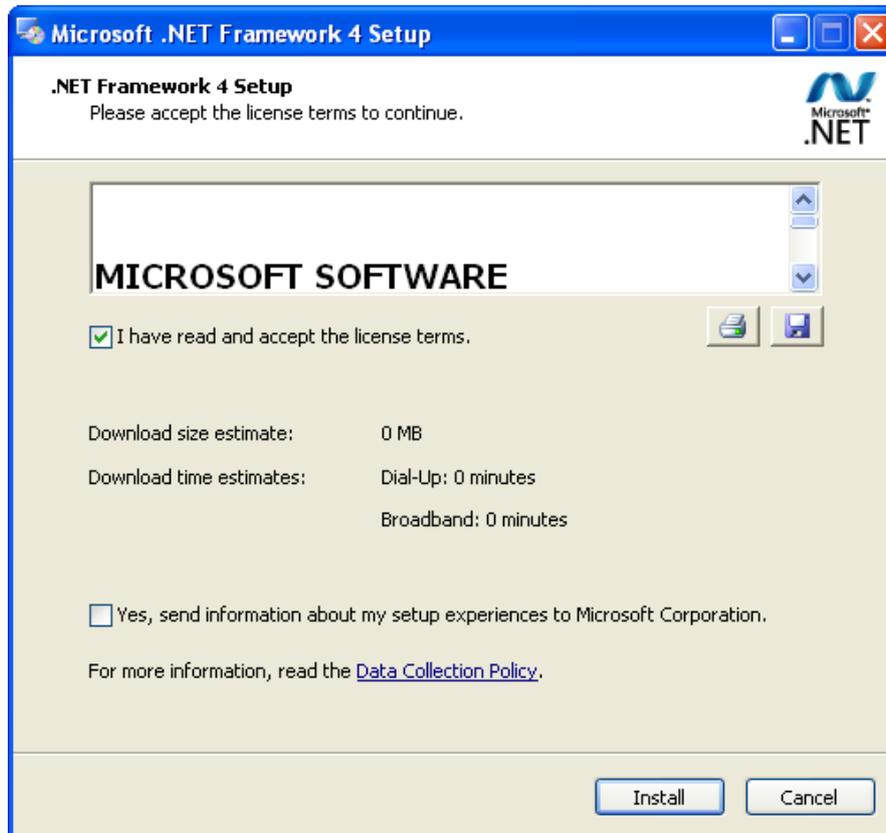


9. The following screen will be displayed, showing the progress of the *.NET Framework* installation.

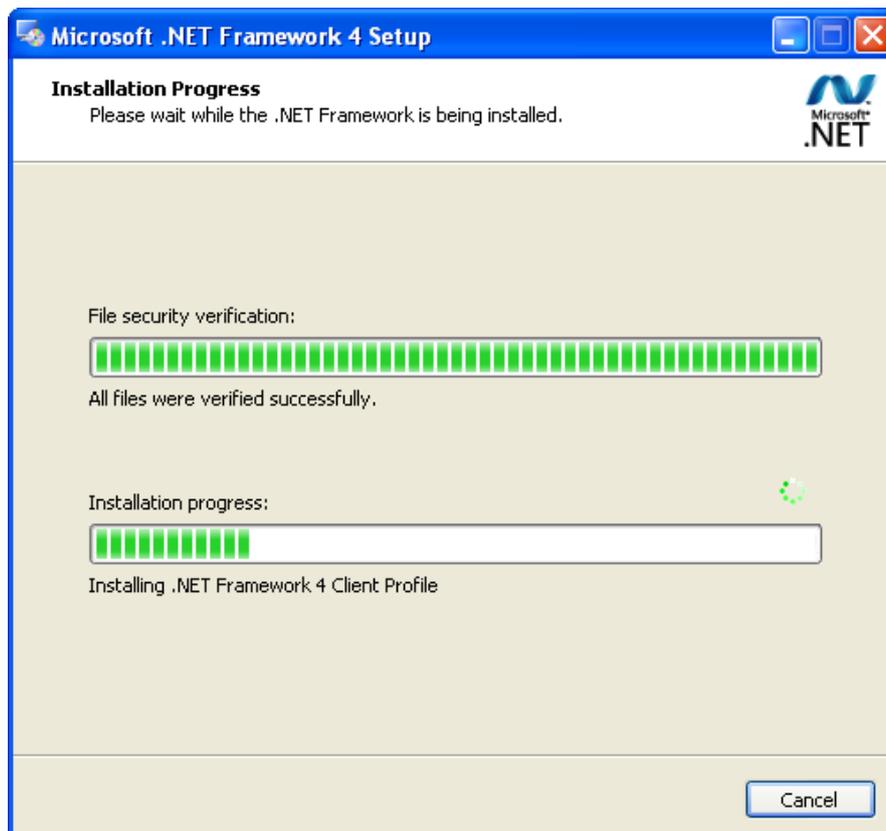

Cholesterol Management Software User's Guide

10. When the *.NET Framework* installation is completed, the interface will be shown below. Click **Finish** to complete the installation.

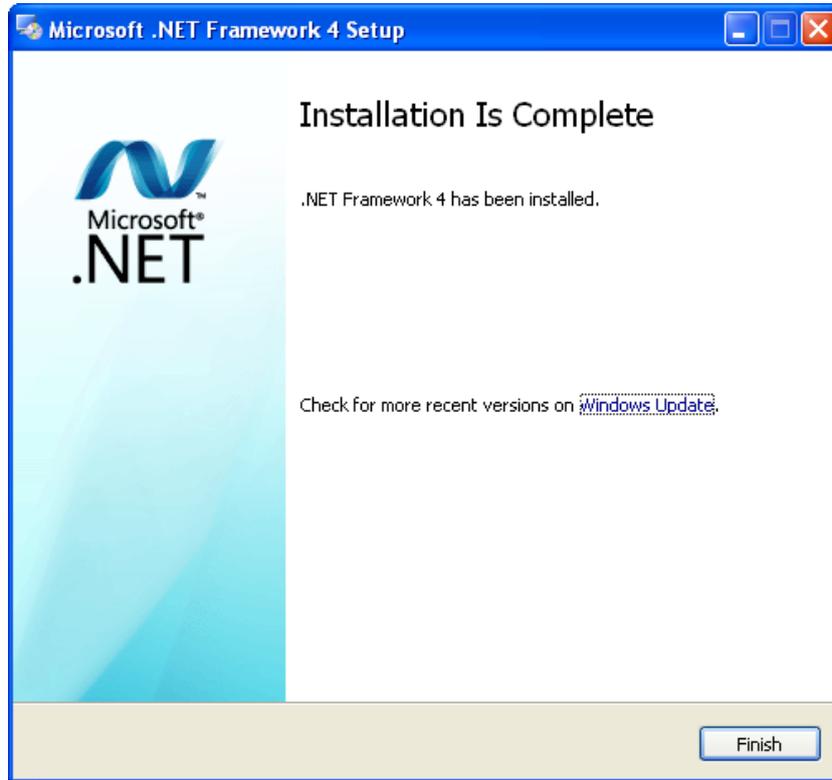

Note: Sometimes the *.NET Framework* installation (step 8-10) is not shown in the software installation, and it is normal. That's because the *.NET Framework* already exists in your computer since it is just a plugin.

11. Click **Finish** to complete the software Setup Wizard.




Cholesterol Management Software Startup


1. Select "CHOL Management Software" from the Program List to run the software, or double click the



following icon on the desktop.

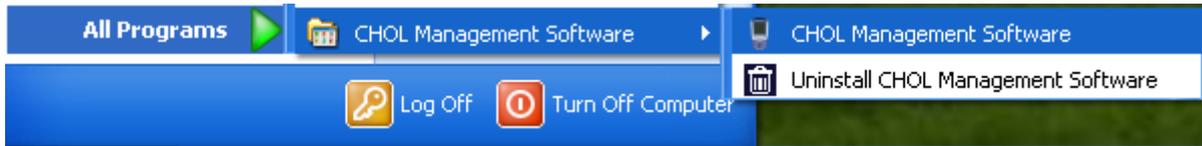

2. The Cholesterol Management Software screen will be displayed as shown below.

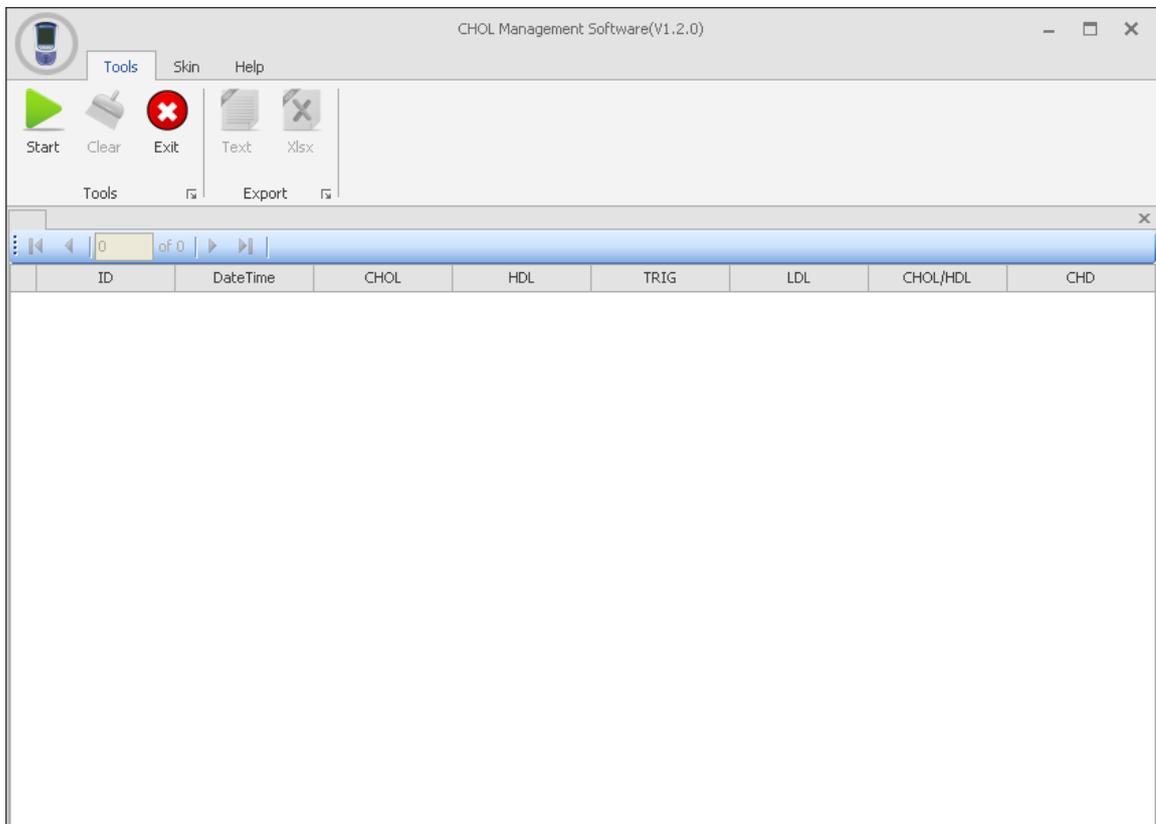


Driver Installation for First Time Use

1. Connect the Cholesterol Meter to a PC using a USB cable. The Found New Hardware wizard will be launched.





Cholesterol Management Software User's Guide

2. Check the PC's Computer Management, the USB program will appear in the device list as shown below.

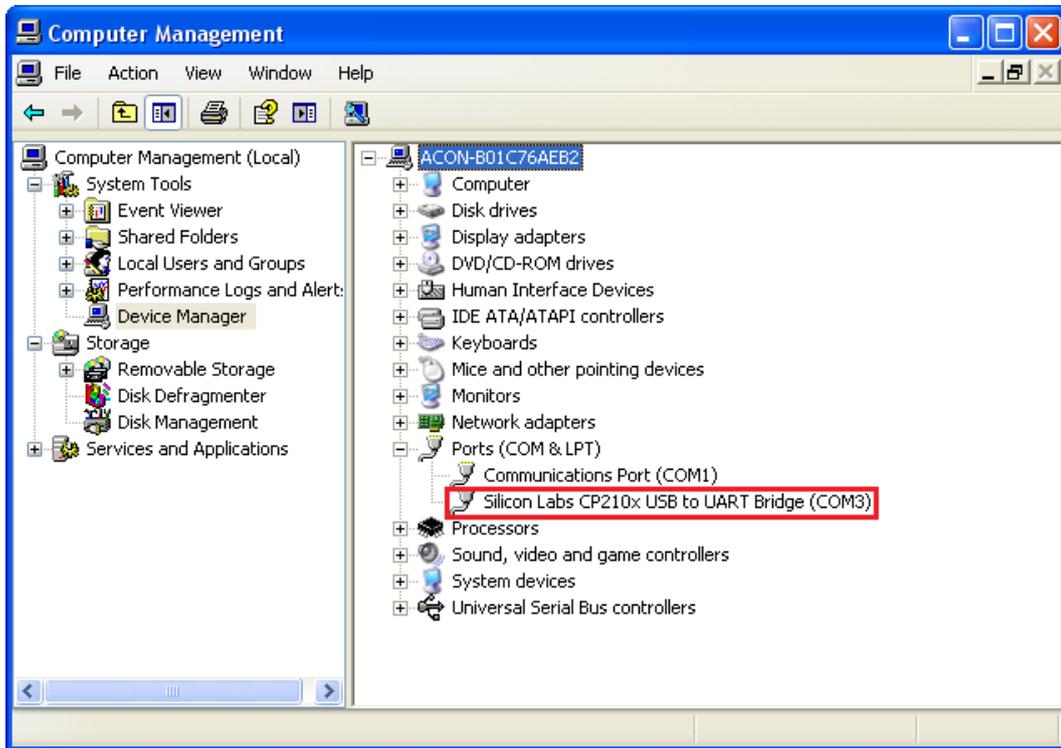


3. If you install the software successfully, the CHOL Management Software and Driver Package will appear in the Control Panel under Add or Remove Programs as follows.

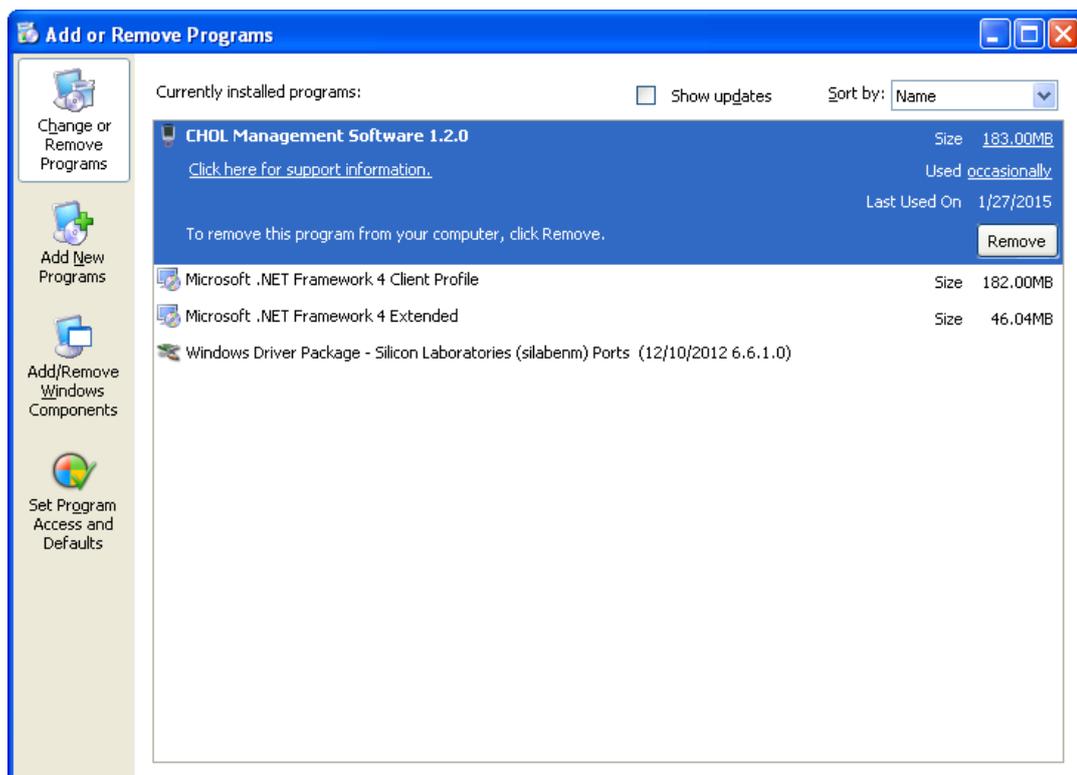

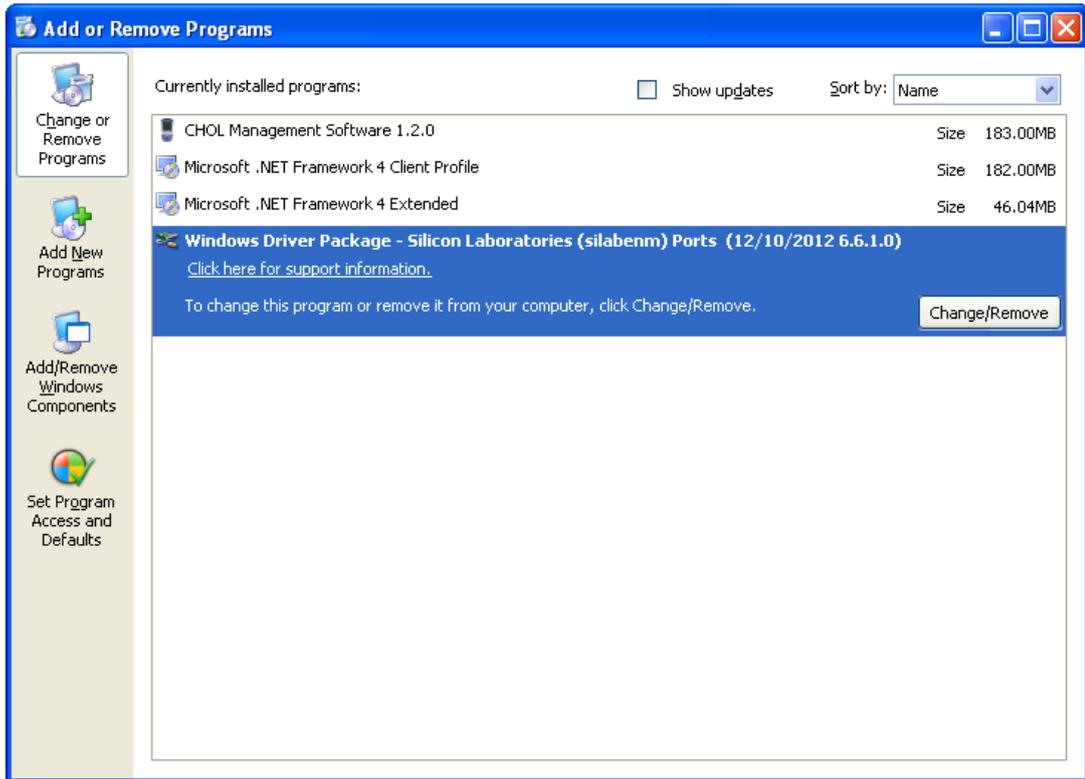

Downloading the Cholesterol Meter Data

1. Use a USB cable to connect the Cholesterol Meter to a PC.
2. Run the software, click the **Start** button to allow the software to start receiving the data. A message will appear as follows.

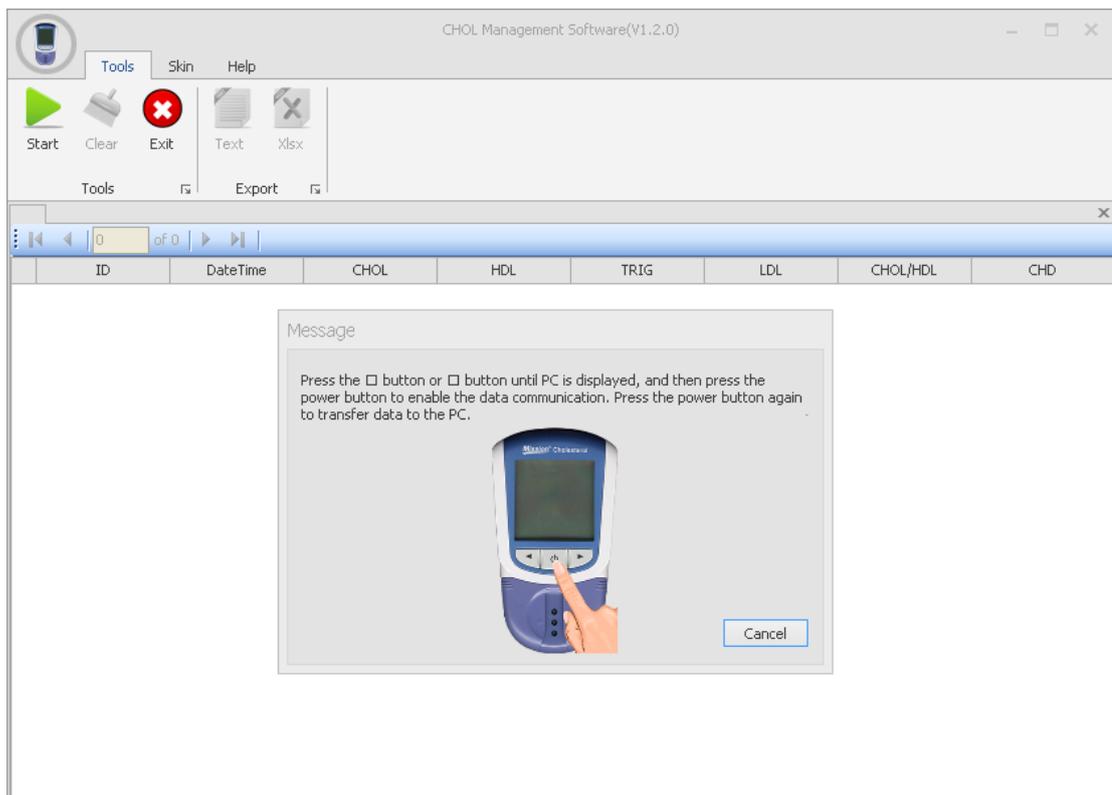


Cholesterol Management Software User's Guide


- Turn on the Cholesterol Meter by holding the power button down until the Set screen appears. Press the ◀ button or ▶ button until **PC** is displayed, and then press the power button to enable the data communication. Press the power button again to transfer data to the PC.

Note: Refer to **Section 8 Data/Communication** in the Cholesterol Monitoring System User's Manual for details.

- The following screen will be displayed, showing the progress of the transfer.

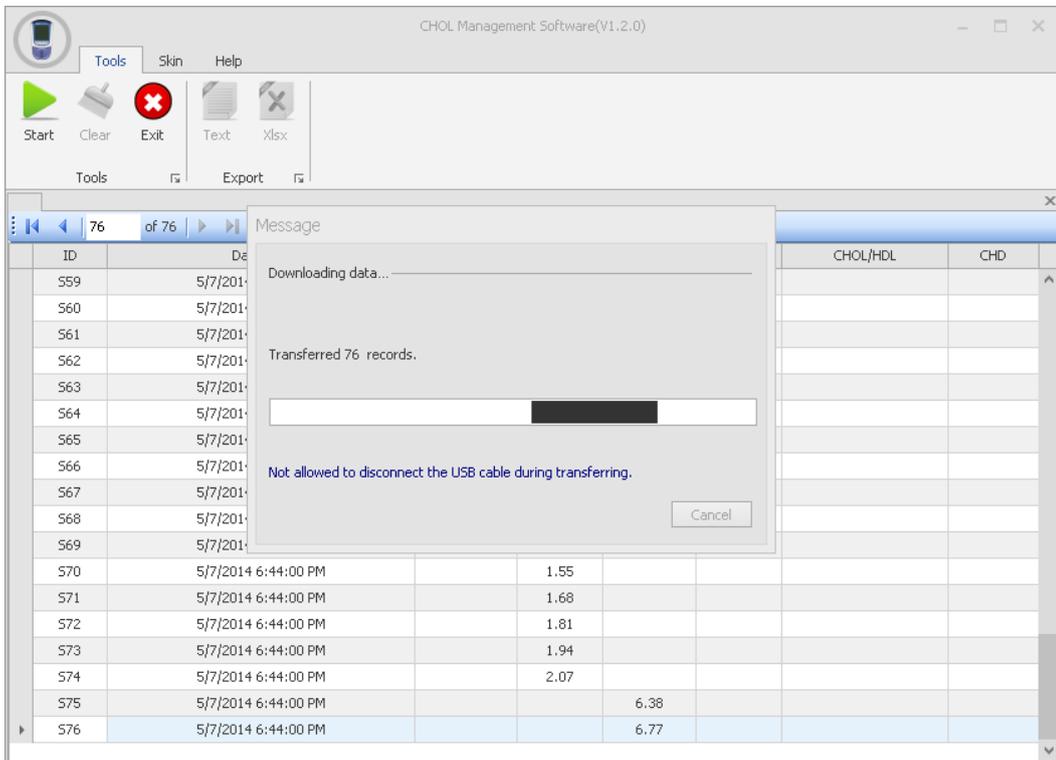


- The records transferred to the PC are shown as below.

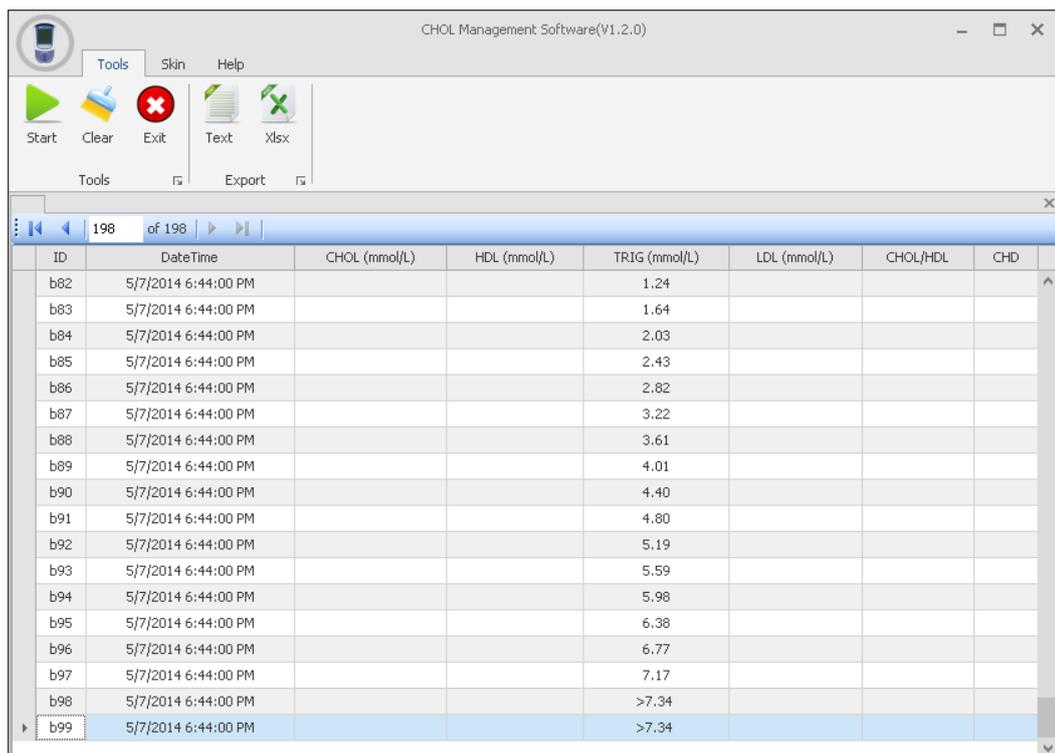


Cholesterol Management Software User's Guide

6. Data transfer will not stop until all records are transferred and the records downloaded will be shown on the software interface. When the transfer is completed, a message displayed as shown below.

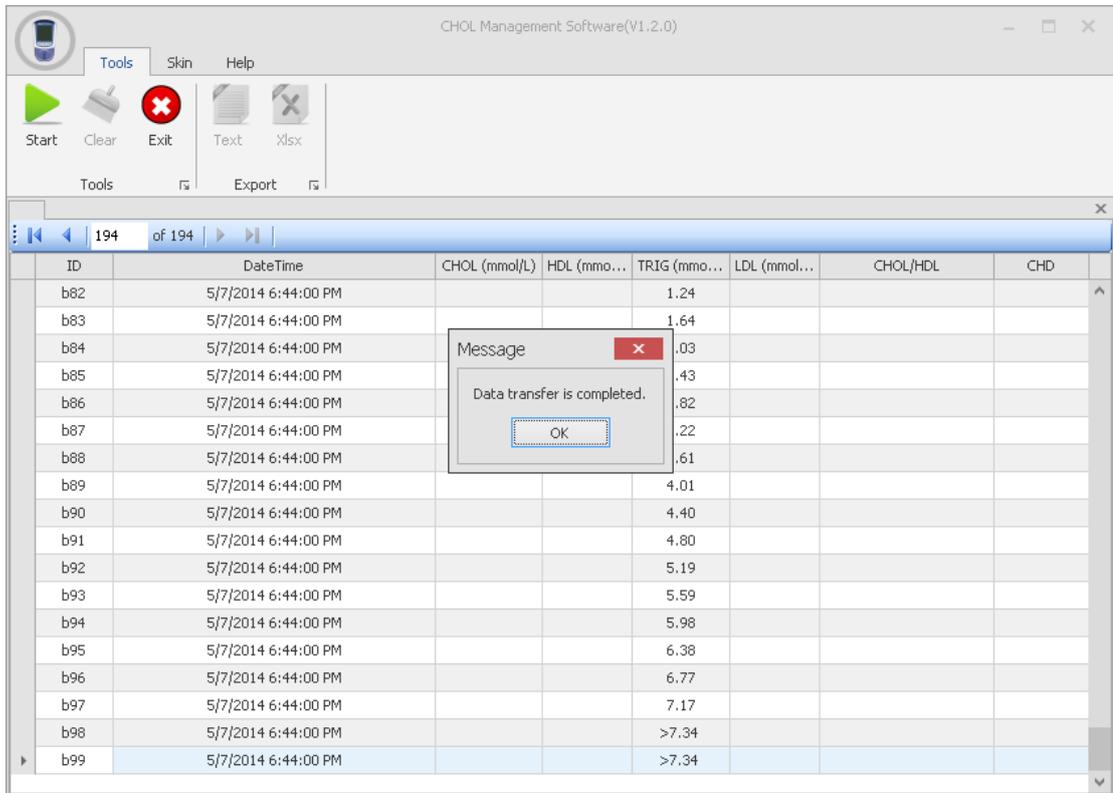


7. Press **Text** or **Xlsx** to export the data as a txt or excel file, and then press save. The file will be saved as shown below. The name of the folder and file can be changed.

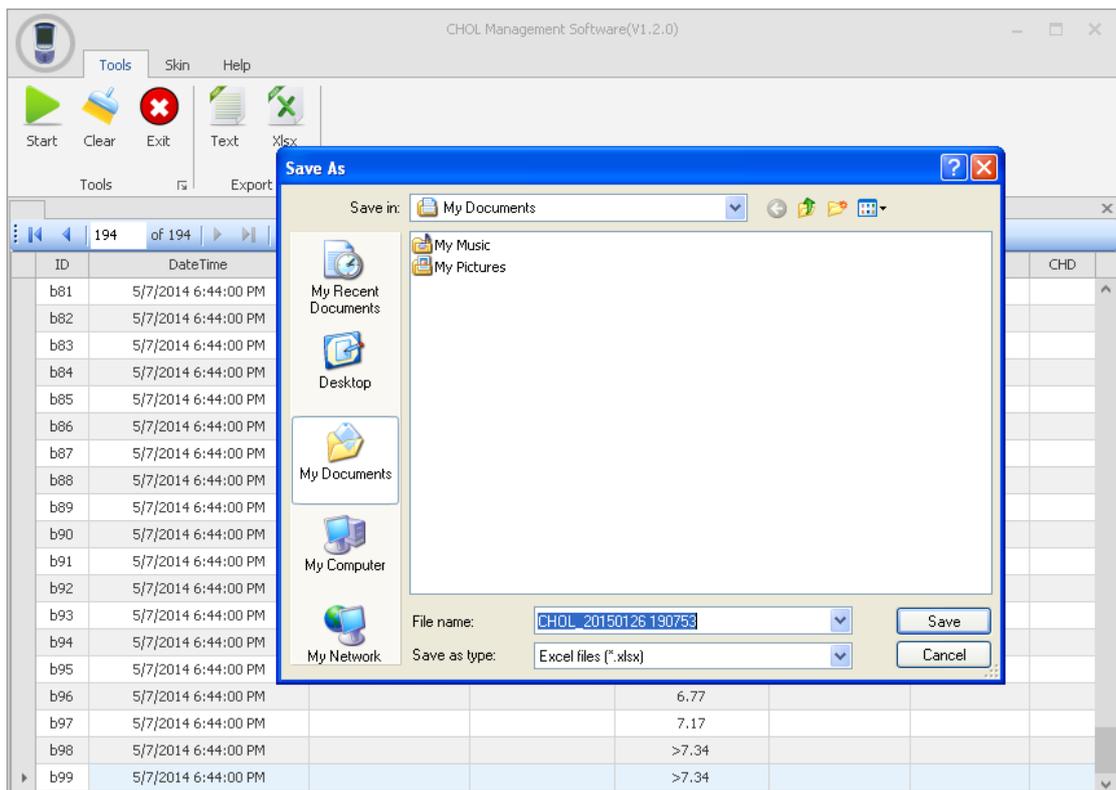


Cholesterol Management Software User's Guide

- Press **Clear** to clear the data on the window or press the **Start** button to clear the data and begin the next data download.
- Different layouts are listed. Click **Skin** to change the layout.

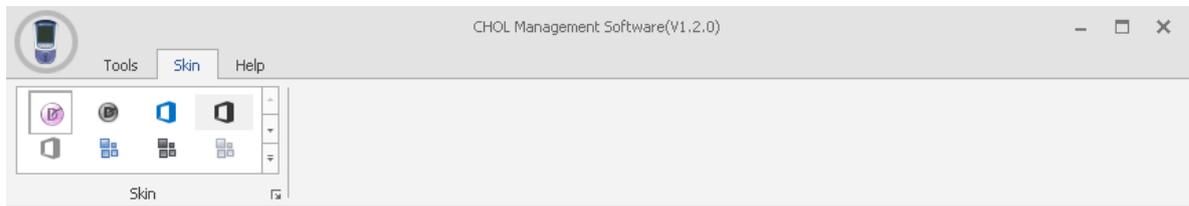

- Go to **Help** to view the user's guide and more information on the software.

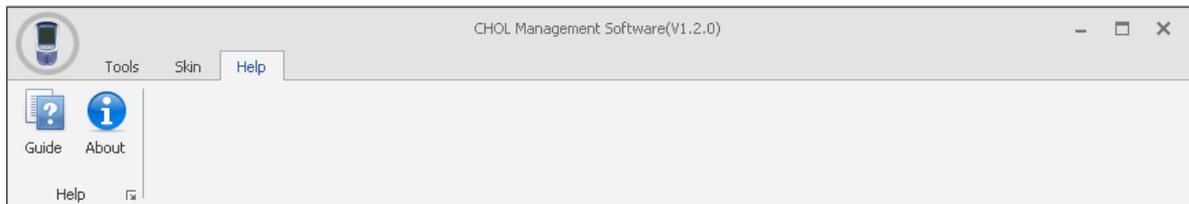


Driver Uninstallation


- Go to the **Start** menu located on the bottom left corner of your computer screen. Select **All Programs** then **CHOL Management Software**. Click on **Uninstall CHOL Management Software**.




- A message will pop up to confirm program uninstallation as follows. Click **Yes** to completely remove the CHOL Management Software and all of its components.

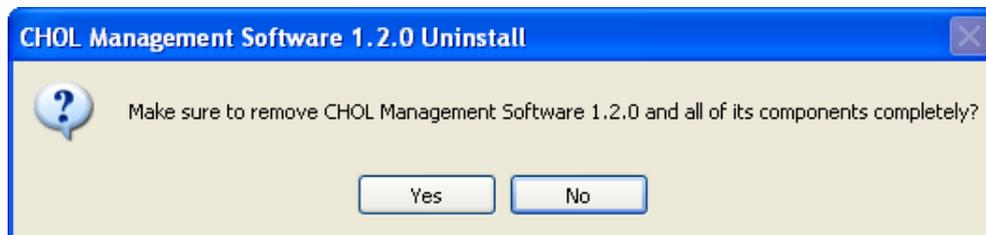

- The following screen will display the uninstallation progress.

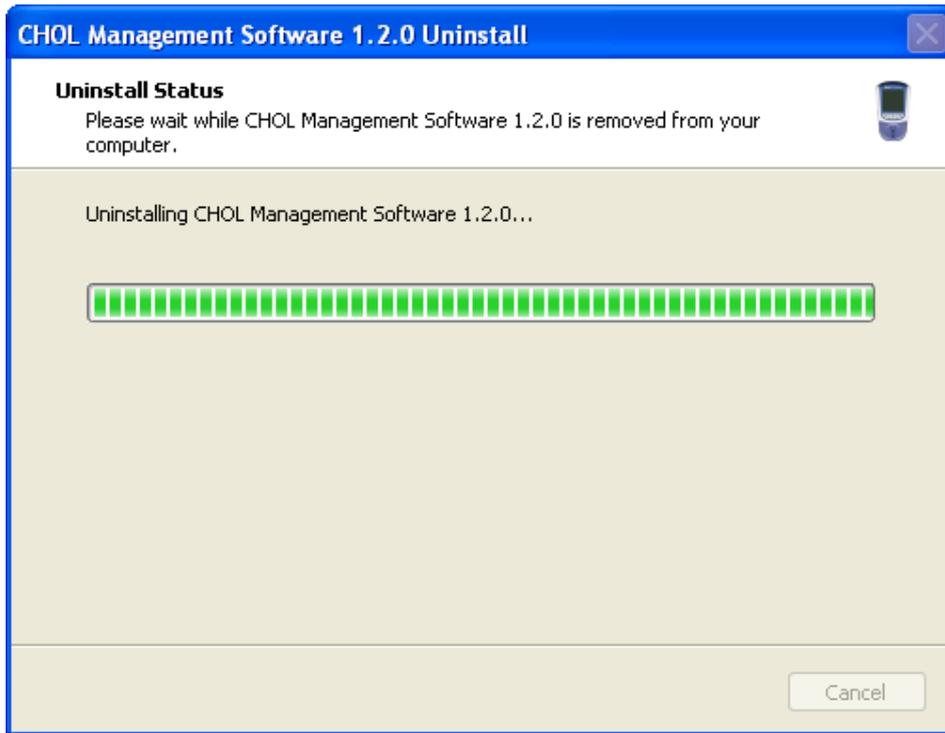

4. Click **OK** to complete the uninstallation process.

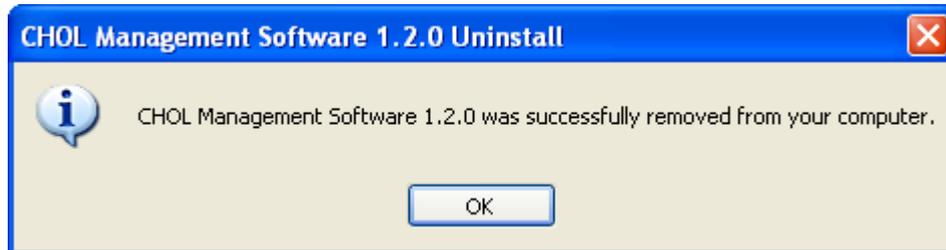

 **ACON Laboratories, Inc.**
10125 Mesa Rim Road,
San Diego, CA 92121, USA

Number: 1150738403
Effective date: 2017-03-22